Demping

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Measured value dB H 17,0 M 18,0 L 16,0 SNR 18,0

- APV** = Assumed Protection Value in dB
- H** = Attenuation for high frequencies
- M** = Attenuation for medium frequencies
- L** = Attenuation for lower frequencies
- SNR** = Single Number Rating

PZT GmbH • An der Junkerei 48 F • D-26389 Wilhelmshaven • EU-Identification Number 1974

Warning

Keep this product away from small children. This packaging contains small parts that could be swallowed. When not in use, please store the earplugs in the supplied box. Do not insert any items in the filter, as they may damage the filter. If you are in a noisy environment, please use the NOIZEZZ earplugs at all times. Failure to follow the procedures outlined in this manual may affect the life and the functioning of the earplugs. NOIZEZZ B.V. cannot be held liable for this.

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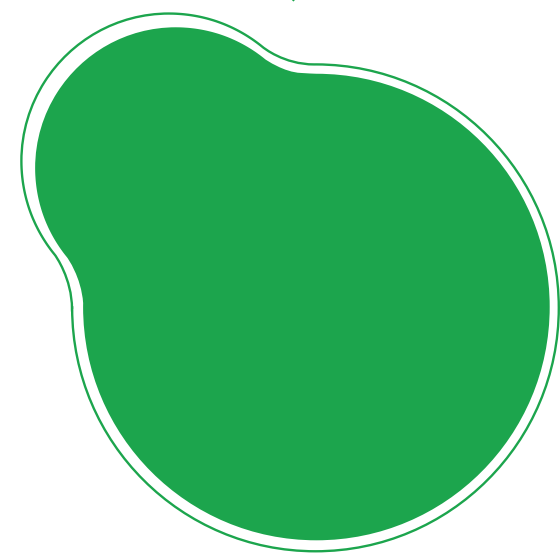
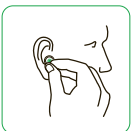
English

How to wear the earplugs:

For the proper functioning of the earplugs, please do as follow:

Pull your outer ear slightly backwards and gently press the earplugs into your ear, until you feel it closes off the ear canal.

Cleaning

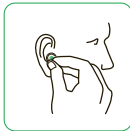
It is recommended to regularly clean the earplugs. Remove any ear wax with a tissue and clean the earplug itself with lukewarm water. The earplugs have to be regularly checked for correct functioning. Chemical substances may have a negative effect on the product. Additional information can be obtained from the manufacturer.

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Hearing protection equipment with filters specifically for going out, festivals and parties. Also great for the catering industry, or for sports instructors in gyms/swimming pools. The unique filters preserve speech intelligibility. These earplugs are one-size-fits-all.

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noizezz

Read the instructions before use and retain for future reference.

For more information please contact:

NOIZEZZ B.V. E-mail: info@noizezz.com

EN 352-2:2002

Manufactured by NOIZEZZ B.V.

Hoofddorp, Netherlands. WWW.NOIZEZZ.COM

Nominal diameter Ø 9 - 13 mm

SNR 18

H 17 M 18 L 16

70002

EAN code:
87330168